

# i+CARE™



## i+CARE™ Midstream Pregnancy Test

**i+CARE™ is more than 99.5 % accurate from the day your period is due. Please read these leaflet carefully before doing test.**

### INTEND TO USE :

**i+CARE™ PREGNANCY TEST** is a single used rapid chromatography immunoassay, which uses specific antibodies to detect  $\beta$ -hCG (Human Chorionic Gonadotropin) at levels as low as 25 mIU/ml or greater in urine to aid in early detection of pregnancy.

**SENSITIVITY :** Greater than 99%  
**SPECIFICITY :** Greater than 99%  
**RESULT TIME :** 3 - 5 Minutes  
**STORAGE :** At 4 C - 30 C (40 F - 86 F), Do not freeze.

### CONTENTS :

- 1 **i+CARE™** Pregnancy test device in Aluminum foil pouch with desiccant.
- 1 Insert instruction

Manufactured by  
**i+MED LABORATORIES CO., LTD. THAILAND**  
 ISO 9001 : 2000 CERTIFIED COMPANY  
**i+MED LABORATORIES** www.imed.co.th

For further information about **i+CARE™** please email to [info@imed.co.th](mailto:info@imed.co.th). Our trained advisor are available to answer your questions and give you confidential advice.

### 1. Doing test.

You can test any time of the day from the day your period is due.

When you are ready to test, remove test stick from foil wrapper. Use the test stick straight away. Take off the cap, and hold Absorbent Sampler pointing downwards in your urine stream for **5 seconds only**.

Keep Absorbent Sampler pointing downwards or lie the test stick flat until the line has appeared in the End Window.

You may wish to replace the cap.

Alternatively, collect a sample of your urine in a clean, dry container and hold just the Absorbent Sampler in the urine for **20 seconds only**.


### 2. Wait...

A line in the End Window shows that the test has worked and it ready to read. This should appear in about 1 minute.

Do not read your result until this line appears.  
 (Refer to question 5 if no line appears.)

### 3. Reading your result

You can now read your result in the Result Window.

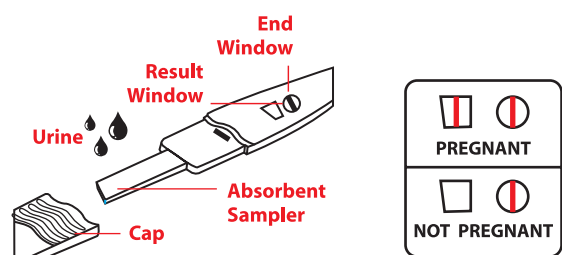

(If you are still unsure of your result see question 6 for full details.)

**IMPORTANT! Read your result within 10 minute of doing test. Disregard any changes after this time.**